





1

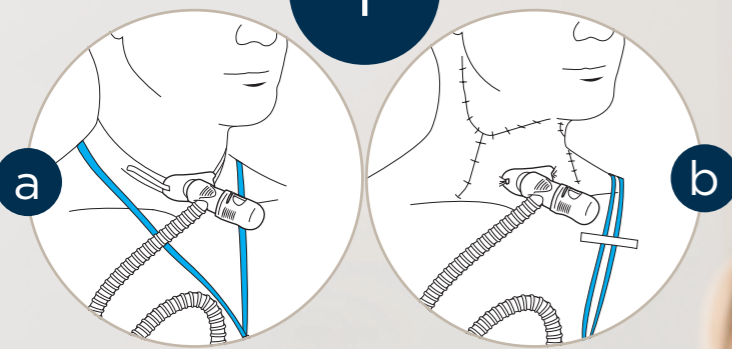

To ensure loading and movement on tracheostomy tube is kept to a minimum, make sure the lanyard is secured properly.

Lanyard position as shown in **1a** is not suitable for patients with damaged skin.



3

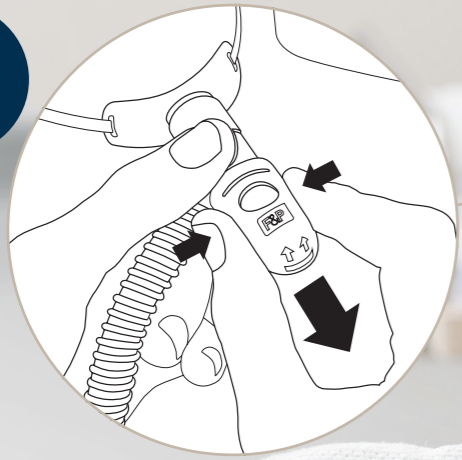

Orientate the sputum guard so that sputum drains downwards. The sputum guard can be removed if required.


2

Connect to the humidification system. Make sure all connections are secure.

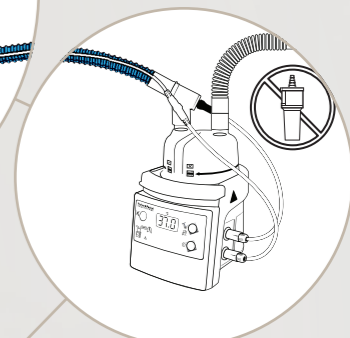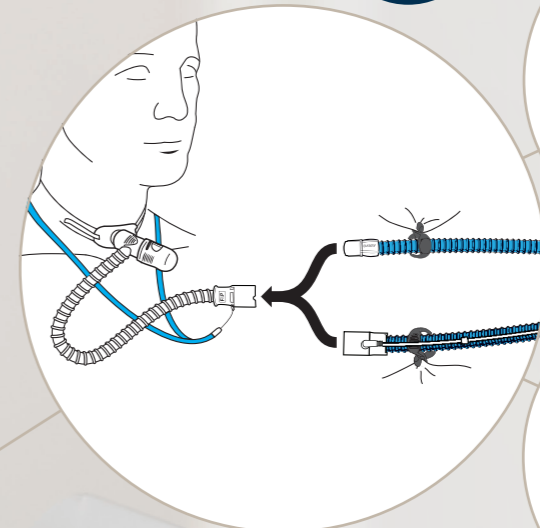

F&P AIRVO

F&P 850 System

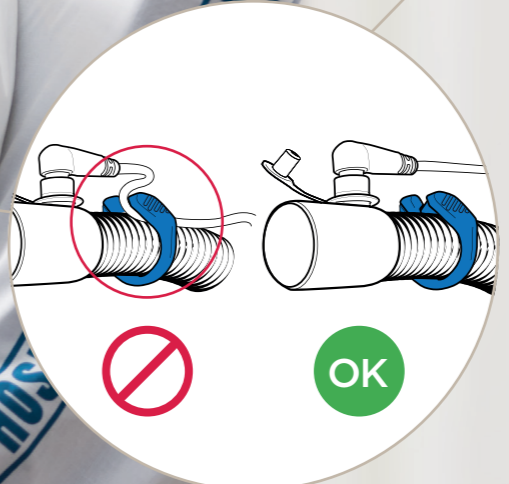

If using an MR850, attach the tubing clip to the breathing circuit but ensure probe cable is not crushed by tubing clip.

i IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.